FREE NOSE® SEA WATER ISOTONIC WITH ALOE VERA AND HYALURONIC ACID - NASAL SPRAY

A proper nasal hygiene is important for adults, children and babies. The excess of mucus in the nasal cavities may increase the microbial proliferation and the excessive dryness of the nasal mucosa.

FREE NOSE® Sea Water with Aloe Vera and hyaluronic acid hydrates strongly nasal cavities thanks to its mechanical action and helps washing and extracting the mucus. The condition of the nasal mucosa is essential in breathing as it filters, humidifies and warms the air we breathe. There are certain occasions when nasal mucosa is altered: rhinitis due to flu, allergies, dry nostril after surgical processes, air conditioners or drugs treatments. Thanks to aloe vera which maintains and prolongs the hydration of the nasal passages and hyaluronic acid which high moisturizing power helps to restore nasal mucosa and promote its normal operation.

FREE NOSE® Sea Water with Aloe Vera and hyaluronic acid maintain the specificity of the vital properties of natural sea water. The sea water comes from the Bay of Cancale in Brittany, renowned for the quality of its water. The Bay of Cancale boasts an exceptional tidal range (12 m) and is swept by strong prevailing currents preventing pollution of the waters. This unique combination ensures that the water constantly cleans itself and is therefore consistently pure. Sampling is performed automatically when the water quality is optimal (after the high tide period). Sampling area is classified as a Class A and is under continuous health monitoring controls (microbiological, chemical contamination, phytoplankton, pH...).

Composition:

Purified water, sea water (sodium chloride 0.9%), Aloe Vera and sodium hyaluronate (0.2%).

Indications:

Proper use of **FREE NOSE® Sea Water with Aloe Vera and hyaluronic acid** in daily nasal hygiene of babies from 1 year, children and adults allows a therapeutic aid in some basic indications such as: nasopharyngitis, obstructive rhinitis and sinusitis, dry nose, hoarseness due to dry mucous membranes, adjuvant in ORL and pediatric treatment.

Instructions for use:

1. Gently insert the tip at the entrance of the nostril.
 2. Press the spray head.
 3. Allow to drain the excess solution in order to extract the mucus, and then blow.
 4. Repeat on the other nostril.
 5. Clean the tip with water and dry.
 6. Cover the nasal applicator with the protective cap until the next use.
- Apply in each nostril as often as necessary.

Instructions to apply the product to the baby:

1. Lay your baby down with head slightly tilted to one side to facilitate the entry of sea water in the nostril.
 2. Pulverize during few seconds pressing the spray head.
 3. Sit up the baby to let the nose drain out. Collect mucus on a tissue.
 4. Repeat on the other nostril.
 5. Gently wipe around the nostrils without putting the tissue deep into the baby's nose. Clean the tip with water and dry.
- Apply to each nostril 2 to 3 times a day.

Contraindications and adverse effects:

- No adverse effects have been described at the recommended therapeutic doses, but if you experience side effects, consult your usual doctor or pharmacist.
- Any serious adverse event related to this product must be declared to the manufacturer, distributor and national health authority in your country.
- At the indicated doses, no interactions are known. Similarly, it is recommended to consult with your habitual doctor if you are taking any medication.

Warnings and precautions:

- For hygienic reasons and to prevent the transmission of pathogens, this container should not be used by more than one person.
- In case of a known or suspected allergy to any of the components of this medical device, it is advisable not to use.
- Keep out of reach of children.
- The source of hyaluronic acid is not of animal origin. It is obtained from bacterial fermentation.

Conservation and expiry date:

- Do not use the product after the expiry date shown on the box and the bottle.
- Maintain in a dry and cool place. Maintain below 40°C and protected from direct sunlight or other heat sources.
- Deposit the bottle and the protective cap in the corresponding containers.
- It is recommended not to discard the box or the leaflet during the use of the product.

Presentation:

Pressurized spray bottle of 120 ml with nasal applicator in SOFT force.

Date of issue: 05/10/2023. Version 03

Knowing your experience with the product will allow us to grow

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