Free Nose® Hypertonic Sea Water - Nasal spray

A proper nasal hygiene is important for adults, children and babies. The excess of mucus in the nasal cavities may increase the microbial proliferation and the excessive dryness of the nasal mucosa.

FREE NOSE® Sea Water hydrates strongly nasal cavities thanks to its mechanical action and helps washing and extracting the mucus, which easily allows the permeability.

FREE NOSE® Sea Water maintain the specificity of the vital properties of natural sea water.

FREE NOSE® Sea Water comes from the Bay of Cancale in Brittany, renowned for the quality of its water. The Bay of Cancale boasts an exceptional tidal range (12 m) and is swept by strong prevailing currents preventing pollution of the waters. This unique combination ensures that the water constantly cleans itself and is therefore consistently pure. Sampling is performed automatically when the water quality is optimal (after the high tide period). Sampling area is classified as a Class A and is under continuous health monitoring controls (microbiological, chemical contamination, phytoplankton, pH...).

Composition:

Sea water and purified water (sodium chloride 2.2%).

Indications:

FREE NOSE® Sea Water hypertonic nasal spray is a medical device that can be used by babies from 3 months, children and adults in case of infection of nasopharynx mucous membranes, runny nose and infection of the paranasal sinuses and snoring caused by dry mucous membranes. Hypertonic nasal spray removes excess fluid from mucous - membrane cells on the basis of the osmotic effect of the hypertonic solution. Besides mechanical cleansing and decongestion of the upper respiratory tract, this osmotic effect helps to reduce swelling of the mucous membranes.

Proper use of **FREE NOSE® Sea Water** in daily nasal hygiene allows a therapeutic aid in some basic indications such as: nasopharyngitis, obstructive rhinitis and sinusitis, dry nose, hoarseness due to dry mucous membranes, adjuvant in ORL and pediatric treatment.

Directions for use:

1. Shake the bottle before use and remove the protective cap. The first time it is used, it may require pressing several times the applicator until the pulverization of the product.
 2. Introduce gently the nasal applicator in one nostril and press the pump to apply the product.
 3. Let drain the excess solution in order to extract the mucus, and then blow.
 4. Repeat the process with the other nostril.
 5. Clean the tip with water and dry.
 6. Cover the nasal applicator with the protective cap until the next use.
- Apply in each nostril as often as necessary.

Instructions to apply the product to the baby:

1. Lay your baby down with head slightly tilted to one side to facilitate the entry of sea water in the nostril.
 2. Pulverize during few seconds pressing the spray head.
 3. Sit up the baby to let the nose drain out. Collect mucus on a tissue.
 4. Repeat on the other nostril.
 5. Gently wipe around the nostrils without putting the tissue deep into the baby's nose. Clean the tip with water and dry.
- Apply to each nostril 2 to 3 times a day.

Contraindications and adverse effects:

- No adverse effects have been described at the recommended therapeutic doses, but if you experience side effects, consult your usual doctor or pharmacist.
- Any serious adverse event related to this product must be declared to the manufacturer, distributor and national health authority in your country.

Warnings and precautions:

- For hygienic reasons and to prevent the transmission of pathogens, this container should not be used by more than one person.
- Keep out of reach of children.

Conservation and expiry date:

- Do not use the product after the expiry date shown on the box and the bottle.
- Maintain in a dry and cool place.
- Discard one month after opening.
- Deposit the bottle and the protective cap in the corresponding containers.

Presentation:

FREE NOSE® Sea Water hypertonic nasal spray 30ml without propellant gas is available in two formats:

- Babies from 3 months.
- Adults and children from 6 years.

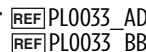
Recommendations:

For babies, we recommend the combined use with the **FREE NOSE® Nasal Aspirator**.

Knowing your experience with the product will allow us to grow

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